**DISTRIBUTION CHAIN PARTICIPANT / IDENTIFICATION CHART**

 PATRON Bohemia a.s. Českolipská 3419, Mělník, CZ ( www.patron.eu )	Product identification label:
Manufacturer—product final inspection / name:  Signature:	

Importer / EU-distributor : (name, address, contact info)	Importer / EU-distributor (stamp):
--	------------------------------------

Point of Sale / Service partner : (name, address, contact info)	Point of Sale / Service partner (stamp):
--	--

Date of sale (day, month, year):  . . .	Name (block letters), phone number, stamp and signature of the seller:
---	--

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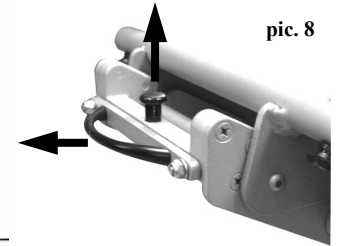
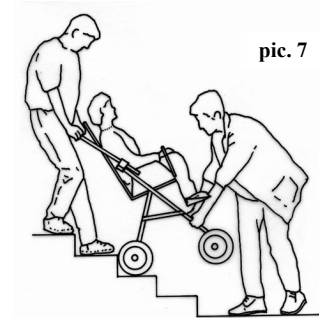
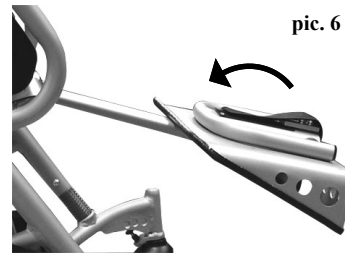
**INTRODUCTION:**

Dear customers, read this document carefully before using this product for the first time. Follow all safety instructions stated in this document. If you have any questions about the product and its proper use, contact professional staff or a service centre. Report any serious adverse event related to the product, or any deterioration in the user's health in connection with the use of the product, immediately to the official distributor of the product in your country or to the manufacturer of the product. For latest safety and/or recall information for this product, as well as the product declaration of conformity, please contact the official distributor of the product in your country or the product manufacturer. Keep this document in a safe place for further reference.

..... **WARNING, CAUTION:**

- ① **CAUTION! Any load attached to the handle (bag, ...) affects the stability of the stroller!**
- ② **The side padded frame covers must be used properly all the time!**
- ③ **Recommended tire air-pressure: 180 kPa**

.....**maximum load.**

**WARNING! Keep fingers out of any moving parts when operating with the stroller.**

**WARNING! Never leave your child unattended! Even if the belts and brakes are in use.**

**WARNING! Check before use that all locking devices are properly locked and all safety equipments are properly used.**

**WARNING! Check before use that seat unit is properly locked in the chassis interface.**

**WARNING! The seat unit is not suitable for a child younger 6 months.**

**WARNING! This product must not be used while jogging or in-line skating.**

**WARNING! The seat unit must not be used in motor vehicles as the car seat.**

**ATTENTION! Read this manual carefully before the first use of the stroller, as well as keep it for further reference. Safety of a child might be in danger in case you will not follow the instructions and safety notices mentioned I the user guide.**

**PRODUCT - DESCRIPTION, FEATURES, IDENTIFICATION:**

You have received a product that is determined for versatile everyday usage outdoor, as well as indoor at home. Before using the product for the first time, get to know carefully how to use it, how to handle with it, how works all the product features, to avoid any possibility of injury. In particular, observe the following instructions:

- All users and / or their accompanying persons must be trained by qualified personnel how to operate with the product. Users and / or their accompanying persons are obliged to read and learn in particular about the residual risks of this product, in connection with the safety instructions given in this document.
- The product has been adapted to the individual needs of the user. Additional changes may only be made by qualified personnel.
- We recommend that you check the product customization once a year to ensure optimal product setup for users. Especially for users with changing anatomy (eg body dimensions and weight) it is recommended to adjust at least once every six months.
- Your specific product may differ from the variants shown here. In particular, not all optional accessories de-

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scribed in this document are mounted on your product.

- The manufacturer reserves the right to make technical changes to the design described in this manual.

### Product features:

This product is classified as a non-invasive active therapeutic device of class 1. The product is intended for short-term and long-term use. The product is controlled by an accompanying person and is used to transport children with the help of another person. The product is intended for use on a solid surface in interiors and exteriors.

The product is used by children with temporary or permanent limitation of the ability to walk, unable to walk or keep standing. The product is suitable for users whose anatomy (eg body dimensions and weight) allows the use of this product for the given purpose and whose skin is intact. The product must only be used in conjunction with the accessories specified in the order form.

The manufacturer of this product assumes no responsibility for combinations with medical devices and / or accessories from other manufacturers. Only product combinations evaluated in terms of their efficiency and safety are permitted. This product and accessory combination must be permitted and stated in the product combination agreement.

Indications: mild to severe or complete restriction of the user's movement.

Contraindications: Absolute contraindications are not known. Relative contraindications - lack of physical or mental preconditions, significant surface damage of surface tissue (eg skin, etc.).

Accompanying phenomenon: Exceptionally, the following accompanying phenomena may occur when using the product - pain in the neck muscles, pain in the muscles and joints, circulatory disorders, risk of pressure sores. In case of these accompanying phenomena or problems, contact your doctor or therapist.

### Product identification:

Information on the product label:

	Global trade identification number. According to GS1-standard = UDI-DI – Unique Device Identifier - Device Identifier. .
	Serial number. According to GS1-standard = UDI-PI – Unique Device Identifier - Production Identifier.
	Producer identification – product version
<b>Mod.:</b>	Product version / text
<b>Spec.:</b>	Product version / colour version
	Producer - address

Date of production: Serial number (first 3 digit) includes production date. („253“ = 253.-months from 01.01.2000 = January 2021)

## SAFETY AND PROTECTION:

- The attendant must be familiar with the instructions of this manual.
- Professional medical advice regarding the suitability of this product must be sought before use.
- Always make sure, before departure, the child is comfortable, and that clothes and rugs etc, are tucked in, so they do not fall in the wheels.
- Ensure that child is clear of moving parts before making adjustments, to avoid any injury.
- When angle-positioning and adjusting the seat and backrest act accordingly to avoid sudden uncontrolled movements.
- Change of the seat direction must be done without the occupant and with empty seat unit.
- The buggy MUST NOT be used as a seat for a child while traveling in motor vehicle or bus if the vehicle is not equipped for that and if the buggy is not crash tested model according to the requirements of ISO 7176-19 or additionally of ANSI/RESNA WC-4:2012 Section 19 Annex A.
- Do not allow the child to climb unassisted into and out from the stroller.
- Always apply the parking brakes, when getting in and out the stroller.
- The stroller is not designed to be driven down the stairs.
- Ensure yourself that the fixing accessories, safety bar and other equipment is fitted and adjusted correctly.

- Do not stand on the footrest.
- The brakes must be engaged at all times when the stroller is not moving.
- Getting over the curb practice firstly with an empty stroller.
- The attendant must take precautions against the fire risk. In buildings, check the location of fire exits and ensure procedures are understood.
- Do not let children play with or hang on the stroller.
- During the drive keep full attention, to avoid any dangerous situation or collisions.
- Do not overload the stroller. Overloading of a chassis, seat, bag or basket causes reduction of stability of the stroller.
- Make the full attention, when manipulate with the stroller, not to injure the child or yourself.
- If it is possible, always use drive-up ramps or lift to overcome any access restrictions.
- Always lift the stroller in two persons.
- Never transport the stroller when you and/or assistance are not sure if you can transport the stroller with the child safely.
- When you drive the stroller on the road, follow the road traffic regulations.
- Do not use the stroller for any other purposes then it is designed for.
- The stroller is designed to carry only one child.
- Do not use any other than original PATRON accessories designed for this product.
- When the stroller is folded and unfolded, the user guide must be followed carefully, to avoid any damage on the construction what could affect the product stability.
- In case of apparent defect, which limits product's safe using (e.g. damage of chassis, part falls away or other optically visible defects, cracks etc.) it is necessary to stop using the product immediately and contact your local official distributor or service centre.
- Company PATRON does not bear any responsibility for the damages caused by using other than original PATRON accessories supplied by this company.
- Company PATRON does not bear any responsibility for the damages and accidents caused by inappropriate use of this product.

## PREPARATION FOR USE, USE:

### 1) Undercarriage:

#### Opening the undercarriage:

Place folded stroller on wheels. Unbutton and release the transport safety-catch on frame.. To open the undercarriage stay between the rear wheels and hold handle tightly. Move handle up and backwards until the handle folding release is fully locked in its end position. To check the locking position, move upwards the handle to ensure that both mechanism are fully locked (the undercarriage must not start folding). The undercarriage is assembled.

#### Folding the undercarriage:

#### Chassis-and-Seat Interface (CSI):

##### Taking off the seat:

Apply the brake. Release the interface lock safety pin and hold it. Simultaneously pull and unlock the seat interface locking release. Lift the seat.

##### Fitting the seat in the driving direction

Place the seat exactly into frame seat fittings. FIRSTLY always start to insert just a side of the seat into the FREE interface socket (without locking pins). By pushing down the second side, the seat will be locked automatically (you will hear the locking sound). Finally check the seat proper locking by slight lifting of the seat unit. Seat must not separate from the interface frame.

##### Fitting the seat against the driving direction:

To place the seat against the driving direction follow the same procedure like in previous chapter except of turning the seat 180° to be facing backside to front.

#### Parking Brake:

To engage the brake, press brake lever pad on both sides fully till the end position (pic.9). To disengage the parking brake, lift both brake lever pads.

#### Swivel lock:

If needed, the front wheel forks can be locked against vertical rotation. To lock the castor forks lift and turn the swivel lock pin by 90° on both sides and let it to snap in in the designated hole on the castor fork.

#### Handle height adjustment:

To change the handle height position press the black plastic buttons on the handle hinges on both sides simultaneously, set the handle to desired position, release buttons. Handle will lock in new position automatically, you will hear the locking sound.

#### Safety Bar:

To take off the safety bar press pin release on both sides simultaneously and pull the bar upwards to remove it. To

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insert the safety bar, both pin releases have to be pointed ahead. Press slightly both press pins and insert the safety bar into the tube inlays at once until the pins are locked. Check the correct position of safety bar by slight pulling up, safety bar must not lift.

### 2) Stroller in operation:

#### Getting over the curb stones and low obstacles:

*Down* - Drive the front wheels to the edge of the curb. Hold the handles firmly and tip the stroller backwards and balance on the rear wheels. Drive the rear wheels to the edge of the curb, lower the stroller down the curb slowly on its rear wheels.

*Up* - Drive the front wheels close to the curb. NEVER bump the front wheels into the curb stone!. Hold the handles firmly and tip the stroller backwards and balance on the rear wheels. Drive forward and place the front wheels on top of the kerb. Position the rear wheels at the curb tone bottom edge and lift to the upper level.

#### Manual transport of the stroller:

- make use of ramps and lifts when they are available
- if there is no alternative and steps should not be avoided at least one trustful assisting person must be available to give assistance
- do not attempt to take the stroller up or down over the number of steps unless you and your assisting person are not sure to have the full control during the manipulation as well as he or she can absorb the weight of its occupant and stroller.
- take the rests in regular intervals, if you are tired
- occupant hands must be inside the seat unit during the manual transport time
- NEVER lift the stroller holding the safety bar or footrest!

#### Going stairs downwards:

Tightly hold the handle and position the stroller squarely with the front wheels on the top step. Tip the handle backward and push the stroller forward until the rear wheels are at the end of top step. Assisting person stands in front, holding the stroller by gripping the front frame. Lift the stroller by both persons at once. Carry the stroller down the steps. Continue step by step to the bottom surface (pic.7).

#### Going stairs upwards:

Tightly hold the handle and position the stroller squarely with the rear wheels against the bottom step. Tip the handle backward and push the stroller forward until the rear wheels are at the end of bottom step. Assisting person stands in front of stroller, holding the stroller by gripping the front frame. Lift the stroller by both persons at once. Carry the stroller upstairs. Continue step by step to the top surface .

### 4) Standard equipment

#### Basket below the stroller :

*Attaching* – Attach the baskets centre part over the undercarriage middle cross bar using fasteners. Fix it with the poppers. Attach the baskets rear part, interlace the fasteners through the side loops placed on the rear frame profiles. Fix it with the poppers. Attach the baskets front part over the undercarriage front curved cross bar using fasteners. Fix it with the poppers .

*Detaching* – unfasten and release all fasteners from the anchoring points.

### 5)Optional equipment:

We offer a wide range of accessories and fixations. The corresponding instructions for use can be found separately in a brochure.

## MAINTENANCE, PRODUCT DISPOSAL:

The stroller has been designed to the high standard and with correct usage and maintenance will provide many years of trouble free service.

- make sure that you are familiar with the operation of the stroller. It should always open and fold easily. If it does not, do not force the mechanism - stop and read the instructions
- regularly inspect handle folding release, safety bar fittings, locking devices, catches, footrest, joints fixings and handle release safety clip to ensure they are secure, clean and in full working order
- brakes, wheels, tyres and upholstery are subjects to wear and should be replaced if necessary by a service centre.
- If you have inflatable tires on the stroller, check its inflation regularly. Allowed tire unevenness on the perimeter of tire is +/- 2 mm
- if the stroller is damaged or in need of any repair, only authorised service centres are allowed to provided this services.
- only spare parts supplied by company PATRON has to be used for repairs.
- should parts of your stroller become stiff or difficult to operate, ensure that dirt is not the cause, apply a light application of a spray lubricant e. g. WD 40 (do not use oil or grease). Ensure that the lubricant penetrates between the components but do not overspray as this may encourage the accumulation of dirt. Wipe off any spray excess. Regular lubrication of moving parts will extend the life of your stroller and make opening and folding easier.
- it is recommend to use rain cover when it rains (it is not standard equipment of the strollers)
- do not expose the stroller to the long-acting intensive solar radiation
- textile components of stroller and/or accessory must be used and stored in the way that the transparent plastic

- parts are not folded or turned tightly. Those can be damaged if the temperature is 5°C below zero.
- wheels, plastic and metal parts may be sponged clean with warm water and mild detergent.
- the seat fabric must be hand washed only at maximum temperature of 40°C, do not use laundry dryer.
- never clean with abrasive, ammonia based, bleaches and alcohol based cleaning compounds.
- after exposure to damp conditions, dry the product off with a soft cloth and leave fully open in warm and dry environment.
- store your stroller in a dry, safe place with recommended temperature 5-20°C.
- it is recommended that the stroller is checked by an approved service centre every 12 months
- We recommend that after 24 months visit a service department.
- Check and tighten the screw connections regularly.
- If there are used control cables (brakes, telescopes, tilt), check it regularly and, if necessary, set up and oil them.

Product lifetime: The expected product lifetime is 5 years. The expected product lifetime was used as a basis for product development and the definition of requirements for the use of the product for the given purpose. This data also includes maintenance requirements to ensure the required efficiency and safety of the product.

Disposal instructions: For disposal, the product must be disposed of in accordance with the local valid environmental protection regulations in the country, where the product was used.

## WARRANTY CONDITIONS:

- On this product is applied 24 months warranty from the date of sale. The warranty is applied for material and manufacturing defects.
- The product has to be used entirely for the purpose it is designed for and the instructions in the user guide must be followed.
- The product has to be properly used, maintained, cleaned and stored in accordance to the user guide instructions.
- Do not pull down the identification tag. It has to be readable and undamaged for the whole warranty period.
- The warranty case has to be claimed by the seller, where the product was bought. Properly filled "Warranty sheet" and the proof of purchase must be declared by the warranty application. Every defect should be claimed immediately
- The claim procedure can be started when the device is delivered complete, in original packaging, carefully packed and secured against damage. The owner is committed to deliver the device clean in accordance to the general hygienic measures. The basic maintenance and cleaning cannot be expected or even required when claimed.
- The warranty does not cover: failures caused by the overloading of the products, accident, forced damage, unauthorized changes made on the product and/or its accessories, natural wear and tear of parts, especially upholstery dirt and damage, wear of plastic parts, brakes, tyre profile and tyre puncture, etc.
- The warranty is terminated whenever the following conditions arise: the user fails to follow the instruction mentioned in this user guide and manufacturer's storage and maintenance suggested procedures, or whenever the product is serviced by any other party except of authorised service centres, or when the identification label is not stucked on the stroller and also in the warranty sheet.
- The warranty period cannot be interchanged with using period, that means for the lifetime period when correct use and maintenance of the product in accordance to the user guide are fulfilled.
- The producer nor the importer is not liable for any losses and injuries caused by manipulation and usage of the product in conflict with the "User Guide" instructions.

The manufacturer, PATRON Bohemia a.s., reserves the right to make changes to the products.

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